1. DESCRIPTION ET INDICATIONS

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Bibliographie:

S. Guero, "Benefits of an unguinal prosthesis in traumatology and reconstructive surgery of the nail", Techniques in hand and upper extremity surgery, 2014 Mar;18(1):20-4. doi: 10.1097

1. DESCRIPTION AND INDICATIONS

The unguinal prosthesis are high density polyethylene lozenges, curved to adapt well to the nails shape. These prostheses are used in case of extraction of the unguinal lozenge. These devices are used in the treatment of unguinal dystrophies from traumatic, infectious or congenital origins. They serve to replace the nail and facilitate regrowth. They are used in emergency when avulsed nail is no longer available, cold to prevent the removal of a healthy nail or reconstructive surgery. These are **USED ONE TIME** products.

All of our standard products have the CE label since 1998

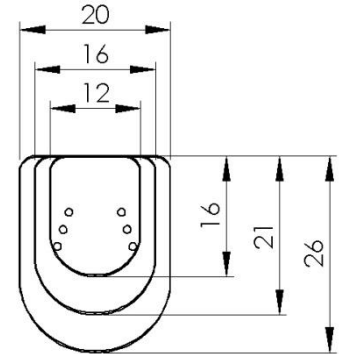
2. COUNTER-INDICATIONS

- Sensitivity to the material,
- All infections which may compromise the implant's function.

3. SIDE-EFFECTS AND POSSIBLE COMPLICATIONS

The patient should be informed of the limits and danger inherent in the implantation:

- Precocious and late infections, haematomas, allergies,
- Mechanical complications: prosthetic rupture, etc.
- Pains or discomfort.

4. INSTRUCTIONS FOR GENERAL USE

- **Operative technique**: see the commercial literature.

Choose a prosthesis slightly larger than the nail being replaced in order to enable its later adaptation to the nail bed. For additional information, please consult the documentation available at our website www.medicalex.info

- Attention
- Check that the product corresponds to the indications on the packaging.
- Follow required asepsis directions when removing from packaging.
- Proceed with a visual exam of the implant in order to detect potential deteriorations (marks, pitting, etc). Do not put the prosthesis in contact with instruments which could alter the surface. In case of serious damage or of visible defects, the implant must not be implanted.
- The unguinal prosthesis must be implanted by a surgeon who has acquired the necessary training.
- Dispose of waste and implants that have come into contact with patients according to current regulations.
- : Do not use if the packaging is damaged.
- : Do not reuse.

5. POSTOPERATIVE PRECAUTIONS

Inform the patient about precautions to take during the operative follow-up.

- Be attentive to any sign of pain in the operated finger.
- Treat quickly and efficiently any neighboring cutaneous infections.
- In case of resumption, contact our company.

It's up to the surgeon to give the authorisation of free movements, as well as to define activities. However, any excessive activity which works the operated finger is strongly unadvised.

6. HANDLING-STORAGE

Products which are delivered sterile are packed individually in protective boxes which must not be opened before use. Store products in these boxes.

- : Fragile, handle with care.
- : Easily damage by humidity.
- : Keeping shielded from the sunlight.

7. STERILITY

- : Do not resterilize.
- : Sterilized by irradiation.

8. MEANING OF THE SYMBOLS

REF	Catalog reference	Manufacturer	Consult precautions for use
LOT	Batch's number	Date of manufacture	Limit date for use

For additional information, please contact us or your dealer or consult the videos and documentation available at our website www.medicalex.info

Bibliography:

S. Guero, "Benefits of an unguinal prosthesis in traumatology and reconstructive surgery of the nail", Techniques in hand and upper extremity surgery, 2014 Mar;18(1):20-4. doi: 10.1097

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