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EN INSTRUCTIONS FOR USE: THORAX RULER

1. DESCRIPTION AND ASSEMBLY

The thorax ruler is CE marked. It is composed of 3 elements: a graduated handle and two arms, one fixed to the handle and the other one sliding along the handle. The handle is graduated in centimetres. Measurements are taken between both arms.

Do not assemble elements of different origins. Only use components marketed by our company. If this were not the case, we could not be held liable.

2. MAJOR INDICATIONS

This product is designed exclusively for taking the chest measurements described in the "General instructions for use" paragraph of these instructions for use.

3. CONTRAINDICATIONS

No contraindication has been identified for this device.

4. ADVERSE EFFECTS AND POSSIBLE COMPLICATIONS

Adverse events and possible complications related to this device are the consequences of inaccurate measurements taken with the device. Inaccurate measurements can result in the making of an unsuitable custom-made dynamic compression brace, which can lead to:

- Discomfort
- Redness
- Skin discolouration
- Skin ulceration
- Irritations
- Respiratory problems
- Pain (chest and back)
- Inappropriate correction of the deformation
- Treatment delay
- Treatment discontinuation

5. GENERAL INSTRUCTIONS FOR USE

- Measurements are in centimetres.
- Measurements are read on the graduated handle, on the sliding arm (where the arrow points to, i.e. on the same side as the fixed arm).
- Measure the chest circumference. All the other measurements must be taken with the thorax ruler provided by our company.
- All the measurements should be taken where the deformation is at its largest.
- The various measurements to be taken are:
 - displacement of the carinatum from the centre of the chest (D, see Figure 1),
 - thorax width (TW, see Figures 1 and 2),
 - thorax depth (TD, see Figures 1 and 3),
 - height of the carinatum (HC, see Figures 1 and 4),
 - chest circumference (CC, see Figures 1 and 5).

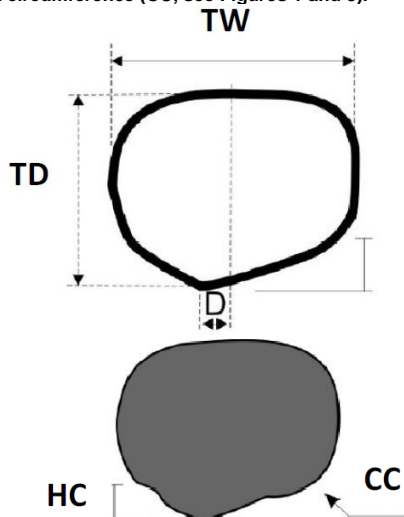
Figure 1 Measurements to be taken

Figure 2 TW: Thorax width

Figure 3 TD: Thorax depth

Figure 4 HC: Height of the carinatum: difference between thorax depth where the deformation is at its largest (TD) and thorax depth where the deformation is at its largest when enough pressure is applied to it to make it disappear (i.e. the pressure for correction).

Figure 5 CC: Example of measurement of chest circumference where the deformation is at its largest

6. PRECAUTIONS AND WARNINGS

Warning

- Be attentive to any signs of patient discomfort when using the product.
- Check that the product corresponds to the instructions given on the packaging.
- Carry out a visual examination of the device in order to detect any deterioration which could compromise its operation.
- Do not put the device in contact with surfaces or substances which could alter its surface, especially the graduations.
- In the event of serious impact or visible defect, the device must not be used.
- In the event of corrective action on an element, our company cannot be held responsible.
- Check the proper tightening of the screws.
- This device must be used by a practitioner who has received the necessary training.

7. HANDLING-STORAGE

Fragile, handle with care.
Store products in their packaging.

8. MAINTENANCE

Disinfect the product before use with a cloth soaked in alcohol. If the product is out-of-adjustment or damaged, send it back to Francemed for repair.

9. MEANING OF THE SYMBOLS

REF	Catalogue reference	Manufacturer	Consult the precautions for use
LOT	Batch number	Medical device	

For additional information, please contact us or your dealer or consult the videos and documentation available at our website www.medicalex.info. Francemed can provide the instructions for use in paper form, at no additional cost, at the latest within 7 calendar days of receiving a request.

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