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INSTRUCTIONS FOR USE: THORAX RULER

1. DESCRIPTION ET ASSEMBLY

The thorax ruler is CE1639 marked. It is composed of 3 elements: a graduated handle and two arms, one fixed to the handle and the other one sliding along the handle.

The handle is graduated in centimeters.

Measurements are taken in between both arms.

Accuracy of measurement: $\pm 3\text{mm}$

Do not assemble components from another origin or manufacturer, only those from our company. If not, we cannot be held responsible.

2. MAJOR INDICATIONS

This product is designed exclusively for the chest measurements described in the present instructions for use.

3. CONTRAINDICATIONS

No contraindication has been identified for this device.

4. ADVERSE EFFECTS AND POSSIBLE COMPLICATIONS

Adverse events and possible complications related to this device are the consequences of inaccurate measurements taken with the device. Inaccurate measurements can lead to making of an unsuited custom-made dynamic compression brace, which can lead to:

- Discomfort
- Redness
- Skin discoloration
- Skin ulceration
- Irritations
- Respiratory problems
- Back and chest pain
- Inappropriate correction
- Delay in completion of treatment
- Treatment termination

5. GENERAL INSTRUCTIONS FOR USE

- Measures are in centimeters.

- Measures are read on the graduated handle, on the sliding arm (where the arrow points to, i.e on the same side as the fixed spout)

- Take a measurement of the chest circumference. All the other measurements must be taken with the thorax ruler provided by our company.

-All the measures should be taken in the place of the greater protrusion.

- Measurements required are:

- Position of carinatum from center of the chest (D, see Figure 1),
- Thorax width (TW, see Figures 1 and 2),
- Thorax depth (TD, see Figures 1 and 3),
- Height of carinatum (HC, see Figures 1 and 4),
- Chest circumference (CC, see Figures 1 and 5).

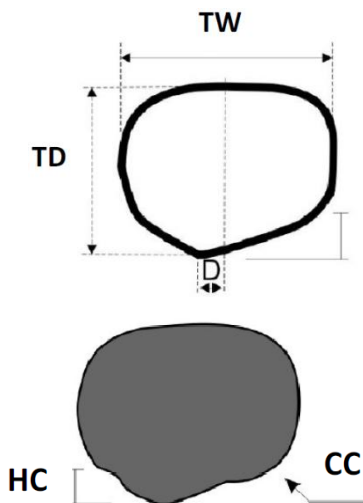
Figure 1 Required measurements

Figure 2 TW: Thorax width

Figure 3 TD: thorax depth

Figure 4 HC: Height of carinatum: difference between thorax depth in the place of the greater protrusion (TD) and thorax depth in the place of the greater protrusion when a pressure is applied on it, enough to make it disappear (i.e. the pressure for initial correction).

Figure 5 CC: Example of measurement of chest circumference in the place of the greater protrusion.

6. PRECAUTIONS AND WARNINGS

: Attention.

- Look out for any sign of discomfort from the patient while using the device.
- Check that the product corresponds to the indications on the packaging.
- Proceed to a visual exam of the device in order to detect potential deterioration that could compromise its ability to function normally
- Do not put the device in contact with surfaces or substances which could alter its surface, especially the graduations.
- In case of serious damage or visible defect, do not use the device.
- In case of corrective actions, our company cannot be held responsible.
- Check tightening of the screws.
- This device must be used by a practitioner who has acquired the necessary training.

7. HANDLING-STORAGE

Fragile, handle with care.
Store products in their packaging.

8. MAINTENANCE

Disinfect before use using a cloth soaked in alcohol.

If the device is out-of-adjustment or damaged, send it back to Francemed for repair.

9. MEANING OF THE SYMBOLS

REF	Catalog reference	Manufacturer	Consult precautions for use
LOT	Batch's number	MD medical device	

For additional information, please contact us or your dealer or consult the videos and documentation available at our website www.medicalex.info Francemed can provide the instructions for use in paper form, at no additional cost, at the latest within 7 calendar days of receiving a request.

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